

HELINI

Coronavirus [Covid-19]

Real-time PCR Kit

[S gene & Human RNaseP]

S gene primer probe falls in del69-70region

For use with Agilent, Bio-Rad, Roche Lightcycler-96, Roche-Z480/Cobas-480, Applied Bio systems [ABI], Thermo-Piko-Real, Rotor gene 5/6plex, Alta-96, Cepheid Real time PCR machines.

Cat.No:8050 – 25/50/100tests

Instruction manual

Intended Use

The HELINI Coronavirus Real-time PCR Kit is an in vitro nucleic acid amplification kit for the detection of novel Coronavirus-2019 specific RNA.

Kit components

Components	Volume Per reaction	25tests	50tests	100tests
One step RT-PCR Master Mix	8 μ l	200 μ l	400 μ l	800 μ l
RT-Taq enzyme mix	2 μ l	50 μ l	100 μ l	200 μ l
COVID-19 Primer Probe Mix [COVID PP mix]	5 μ l	125 μ l	250 μ l	500 μ l
Positive control	10 μ l	100 μ l	100 μ l	100 μ l
Water, PCR grade		1ml	1ml	1ml

Storage

- The kit is shipped on gel ice. Upon arrival, all components should be stored in -20°C. They are stable until the expiration date stated on the label.
- Repeated thawing and freezing should be avoided, as this might affect the performance of the assay.
- If the reagents are to be used only intermittently, they should be frozen in aliquots. Storage at 2 to 8°C should not exceed a period of 5 hours.

Material and instruments required

- Real-time PCR instrument having FAM & HEX channel
- Automatic Nucleic acid extraction system or spin column-based purification kit for the purification of nucleic acids
- Desktop centrifuge having 13000rpm or above with a rotor for 1.5/2 ml reaction tubes
- Centrifuge with a rotor for PCR strips/tubes and 96 well plates
- Optical cap qPCR tubes or strips or 96 wells
- Micro Pipettes (variables)
- Micro Pipette tips with filters (disposable)
- Powder-free gloves (disposable)

[Please ensure that all instruments used have been installed, calibrated, checked and maintained according to the manufacturer's instructions and recommendations.]

Product Use Limitations

- All reagents may exclusively be used in molecular diagnosis.
- The product is to be used by personnel specially instructed and trained in Molecular diagnosis.
- Strict compliance with the user manual is required for optimal PCR results.
- Attention should be paid to expiration dates printed on the box and labels of all components. Do not use expired components.
- Wear protective disposable powder-free gloves, a laboratory coat and eye protection when handling specimens and kit components.
- Avoid microbial and nuclease (RNAse/RNase) contamination of the specimens and the components of the kit.
- Always use RNAse/RNase-free disposable pipette tips with aerosol barriers.
- Use separated and segregated working areas for sample preparation, reaction setup and amplification/detection activities.
- The workflow in the laboratory should proceed in unidirectional manner. Always wear disposable gloves in each area and change them before entering a different area.
- Store positive and/or potentially positive material separated from all other components of the kit.
- Do not open the reaction tubes/plates post amplification, to avoid contamination with amplicons.

- Additional controls may be tested according to guidelines or requirements of local, state and/or federal regulations or accrediting organizations.
- Do not autoclave reaction tubes after the PCR, since this will not degrade the amplified nucleic acid and will bear the risk to contaminate the laboratory area.
- Discard sample and assay waste according to your local safety regulations.

Technical Assistance

For technical assistance and more information, please contact;
0091-9382810333
0091-44-244490433
helinibiomolecules@gmail.com

Product description

HELINI Coronavirus Real-time PCR kit constitutes a ready-to-use system for the detection of Novel 2019 Coronavirus specific RNA using polymerase chain reaction (PCR). It contains reagents and enzymes for the specific amplification of the conserved region of the Novel Coronavirus genome, and for the direct detection of the specific amplicon in FAM channel. In addition, it contains an endogenous control amplification system to identify the sample collection, possible PCR inhibition and RNA purification efficiency.

Specificity

Novel Coronavirus primer and probe have been designed for the specific and exclusive *in vitro* detection of Novel Coronavirus 2019. The target viral gene is Spike protein (S). Sequences in this kit have 100% homology with a broad range of clinically relevant reference sequences based on a comprehensive bioinformatics analysis.

Please note that S gene primer probe region present in the reported 69-70deletion mutation.

Dynamic linear range

The linear range was evaluated by analyzing a logarithmic dilution series of RNA cDNA concentrations ranging from 1.00E+09 to 1.00E+00 copies/ μ l. At least six replicates per dilution were analyzed. The linear range is 1.00E+09 to 1.00E+00 copies/ μ l.

Analytical Sensitivity

The analytical sensitivity is defined as the concentration of RNA molecules (copies/ μ l) that can be detected with a positivity rate of 95%. The analytical sensitivity was determined by analysis of dilution series of quantified Coronavirus specific pDNA from 0.001copies to 10copies/ μ l in triplicates. Under optimal PCR conditions, the analytical sensitivity is 0.35 copy per micro liter.

Note:

RNA Purification

Purified RNA is the starting material for the Real-time PCR assay. The quality of the purified RNA has a profound impact on the performance of the entire test system. It has to be ensured that the purification system used for RNA purification is compatible with real-time PCR technology.

If you are using a spin column-based sample preparation procedure having washing buffers containing ethanol, it is highly recommended to perform an additional centrifugation step for 5min at approximately 17000 x g (~ 13000 rpm), using a new collection tube, prior to the elution of the RNA.

Endogenous control

Human gene [RNaseP] is given as endogenous control. It amplifies a single copy human gene from the test samples. A successful amplification indicates that test sample is properly collected and has its biological property with required number of human cells for PCR.

The Endogenous control primer and probe present at PCR limiting concentrations which allows multiplexing with the target sequence primers. Amplification of the endogenous control template does not interfere with detection of the pathogen even when present at low copy number. The endogenous control is detected through the HEX channel and gives a CT value of 25 +/-9.

Detection Protocol

Things to do before starting

- Before use, all kit components need to be thawed completely, mixed by gently inverting and centrifuged briefly.
- Make sure that Positive and Negative control is included in every run.
- Include 0.5 reaction volume for pipetting error while calculating the volume for total number of reactions.

Components	Volume
One step RT-PCR Master Mix	8µl
RT-Taq enzyme mix	2µl
COVID PP Mix	5µl
Purified RNA	10µl
Final reaction volume	25µl

Negative Control setup [NTC]

Add 10µl of PCR grade water.

Qualitative Positive Control setup

Add 10µl of the Positive control

Centrifuge PCR vials briefly before placing into thermal cycler.
[Note: There should not be any bubbles in the reaction mix. Bubbles interfere with fluorescence detection.]

Programming Thermal cycler

Sample volume	25µl
Fluorescence Dyes	FAM & HEX
Passive reference	None
Ramping rate	Default

Thermal Profile

	Step	Time	Temp
	Reverse transcriptase	20min	50°C
	Taq enzyme activation / Hold	15min	95°C
45 cycles	Denaturation	20sec	95°C
	Annealing/Data collection*	20sec	56°C
	Extension	20sec	72°C

*Data collection/Acquisition	Targets
FAM	Covid-19 RNA - S gene
HEX	Endogenous control

Reading the graph:**Step-1 – Endogenous control Validation**

Select **ONLY test samples** for the endogenous control analysis. Select HEX dye and view the graph of endogenous control amplification. A successful amplification Ct value must be within Ct 25 +/- 9.

This range indicates that test sample has enough cells to perform PCR reaction and NO PCR inhibition in the reaction. Any sample value goes beyond Ct 35 indicates that either sample does not have enough cells OR issues in the purification OR inhibition in the PCR reaction.

Endogenous control will not get amplified in the negative control and Positive control. Ignore a late noise HEX amplification graph in the NTC & PC. In case, any late HEX amplification in the test sample, analyze the graph in log scale mode and there should be minimum 2 Ct difference between the NTC/PC noise and test sample. In Log scale, the test sample starting fluorescence cycle can be observed.

Step-2

FAM-S gene: Select the NTC and Positive control wells, select FAM channel, and view the graph of amplification. The NTC must be flat with no Ct value. If required adjust the threshold value just above the NTC. The PC must be amplified.

Please note that S Gene Primer probe are falling in reported S gene deletion 69-70. No amplification of S gene must be reported as Mutated strain.

Qualitative interpretation of results:

Negative Control	Positive Control	Endo Control	S gene	Interpretation
Negative	Positive	Positive	Positive	2019 Novel Coronavirus RNA detected
Negative	Positive	Positive	Negative	COVID-19 Variant detected
Negative	Positive	Positive	Negative	Not Detected
Negative	Negative	Negative	Negative	Experiment failed – repeat
Positive	Positive	Positive	Positive	Experiment failed – Check the reagent or environment contamination

Limitations

Good laboratory practice is essential for proper performance of this assay. Strict compliance with the instructions for use is required for optimal results.

Analysts should be trained and familiar with testing procedures and interpretation of results prior to performing the assay.

A false negative result may occur if inadequate numbers of organisms are present in the sample due to improper collection, transport or handling. Appropriate specimen collection, transport, storage and processing procedures are required for the optimal performance of this test.

Extreme care should be taken to preserve the purity of the components of the kit and reaction setups. All reagents should be closely monitored for impurity and contamination. Any suspicious reagents should be discarded.

The presence of PCR inhibitors may cause under quantification, false negative or invalid results.

Potential mutations within the target regions of the pathogen's genome covered by the primers and/or probes used in the kit may result in under quantification and/or failure to detect.

As with any research, the HELINI Coronavirus Real-time PCR results need to be interpreted in consideration of all clinical and laboratory findings.

Quality Control

In accordance with the HELINI Biomolecules in house Quality Management System, each lot of HELINI Coronavirus Real-time PCR kit is tested against predetermined specifications to ensure consistent product quality.

Manufactured by

HELINI Biomolecules,

Ohmlina, 26, 2nd Avenue,

Khuthubi Complex, Vettuvankeni,

Chennai 600115, Tamil Nadu, INDIA

www.helini.in

info@helini.in

helinibiomolecules@gmail.com

+91-44-24490433 / +91-9382810333

