

# HELINI

## Coronavirus

### Real-time PCR Kit

[Confirmation two tubes assay]

Instructions for use

**For use with:** Agilent, Bio-Rad, Roche Lightcycler-96, Roche-Z480/Cobas-480, Applied Bio systems [ABI], Thermo-Piko-Real, Rotor gene 5/6plex, Alta-96, Cepheid Real time PCR machines.

**RUO**

**REF**

8054



25



HELINI Biomolecules, Chennai, INDIA

## Contents

Intended use	5
Kit components	5
Storage	6
Material and Instruments required	6
Product use limitations	7
Product description	10
Specificity & Linear range	10
Analytical sensitivity	11
Note – RNA purification	11
Endogenous control system	12
Detection protocol – reaction setup	13
Programming – Thermal profile & Dye selection	14
Reading the graph	15
Qualitative interpretation	16
Copies number calculation	16
Limitations	17
Quality control	18

### Intended Use

The HELINI Coronavirus Real time PCR - confirmation two tube assay is an in vitro nucleic acid amplification kit for the detection of Novel coronavirus-2019 specific RNA.

### Kit components

Components	Volume Per reaction	Number of vials	Volume Per vials
One step RT-PCR Master Mix	8 $\mu$ l	2	200 $\mu$ l
RT-Taq enzyme mix	2 $\mu$ l	2	50 $\mu$ l
RdRp Primer Probe Mix [RDRP PP Mix]	5 $\mu$ l	1	125 $\mu$ l
ORF Primer Probe Mix [ORF PP Mix]	5 $\mu$ l	1	125 $\mu$ l
Positive control	10 $\mu$ l	1	250 $\mu$ l
Water, PCR grade		1	4ml

### Storage

- The kit is shipped on gel ice. Upon arrival, all components should be stored in -20°C. They are stable until the expiration date stated on the label.
- Repeated thawing and freezing should be avoided, as this might affect the performance of the assay.
- If the reagents are to be used only intermittently, they should be frozen in aliquots. Storage at 2 to 8°C should not exceed a period of 5 hours.

### Material and instruments required

- Real-time PCR instrument having FAM & HEX channel
- Automatic Nucleic acid extraction system or spin column-based purification kit for the purification of nucleic acids
- Desktop centrifuge having 13000rpm or above with a rotor for 1.5/2 ml reaction tubes
- Centrifuge with a rotor for PCR strips/tubes and 96 well plates
- Optical cap qPCR tubes or strips or 96 wells
- Micro Pipettes (variables)
- Micro Pipette tips with filters (disposable)
- Powder-free gloves (disposable)

*[Please ensure that all instruments used have been installed, calibrated, checked and maintained according to the manufacturer's instructions and recommendations.]*

### Product Use Limitations

- All reagents may exclusively be used in molecular research.
- The product is to be used by personnel specially instructed and trained in Molecular research.
- Strict compliance with the user manual is required for optimal PCR results.
- Attention should be paid to expiration dates printed on the box and labels of all components. Do not use expired components.
- Wear protective disposable powder-free gloves, a laboratory coat and eye protection when handling specimens and kit components.
- Avoid microbial and nuclease (RNAse/RNase) contamination of the specimens and the components of the kit.
- Always use RNAse/RNase-free disposable pipette tips with aerosol barriers.
- Use separated and segregated working areas for sample preparation, reaction setup and amplification/detection activities.
- The workflow in the laboratory should proceed in unidirectional manner. Always wear disposable gloves in each area and change them before entering a different area.
- Store positive and/or potentially positive material separated from all other components of the kit.
- Do not open the reaction tubes/plates post amplification, to avoid contamination with amplicons.

- Additional controls may be tested according to guidelines or requirements of local, state and/or federal regulations or accrediting organizations.
- Do not autoclave reaction tubes after the PCR, since this will not degrade the amplified nucleic acid and will bear the risk to contaminate the laboratory area.
- Discard sample and assay waste according to your local safety regulations.

### Technical Assistance

For technical assistance and more information, please contact;

0091-9382810333

0091-44-244490433

helinibiomolecules@gmail.com

---

### Product description

HELINI Coronavirus Real-time PCR confirmation two tubes assay constitutes a ready-to-use system for the detection of Coronavirus-2019 specific RNA using polymerase chain reaction (PCR). It contains reagents and enzymes for the specific amplification of the conserved region of the Novel Coronavirus-2019 genome, and for the direct detection of the specific amplicon in FAM channel.

### Specificity

Primer and probe have been designed for the specific and exclusive *in vitro* detection of Coronavirus-2019, The target genes and sequences [RdRp & ORF] in this kit have 100% homology with a broad range of clinically relevant reference sequences based on a comprehensive bioinformatics analysis.

### Dynamic linear range

The linear range was evaluated by analyzing a logarithmic dilution series of RNA cDNA concentrations ranging from 1.00E+09 to 1.00E+00 copies/ $\mu$ l. At least six replicates per dilution were analyzed. The linear range is 1.00E+09 to 1.00E+00 copies/ $\mu$ l.

---

### Analytical Sensitivity

The analytical sensitivity is defined as the concentration of RNA molecules (copies/ $\mu$ l) that can be detected with a positivity rate of 95%. The analytical sensitivity was determined by analysis of dilution series of quantified Coronavirus specific pDNA from 0.001copies to 10copies/ $\mu$ l in triplicates. Under optimal PCR conditions, the analytical sensitivity is 1 copy per micro liter.

### Note:

#### RNA Purification

Purified RNA is the starting material for the Real-time PCR assay. The quality of the purified RNA has a profound impact on the performance of the entire test system. It has to be ensured that the purification system used for RNA purification is compatible with real-time PCR technology.

If you are using a spin column based sample preparation procedure having washing buffers containing ethanol, it is highly recommended to perform an additional centrifugation step for 10 min at approximately 17000 x g (~ 13000 rpm), using a new collection tube, prior to the elution of the RNA.

## Detection Protocol

### Things to do before starting

- Before use, all kit components need to be thawed completely, mixed by gently inverting and centrifuged briefly.
- Make sure that Positive and Negative control is included in every run.
- Include 0.5 reaction volume for pipetting error while calculating the volume for total number of reactions.

Components	RDRP	ORF
One step RT-PCR Master Mix	8µl	8µl
RT-Taq enzyme mix	2µl	2µl
RDRP PP Mix	5µl	---
ORF PP Mix	---	5µl
	<b>15µl</b>	<b>15µl</b>
Thoroughly mix the sample RNA by pipetting up and down.		
<b>Purified RNA</b>	<b>10µl</b>	<b>10µl</b>
Final reaction volume	25µl	25µl

### Negative Control setup [NTC]

Add 10µl of PCR grade water.

### Qualitative Positive Control setup

Add 10µl of the Positive control

Centrifuge PCR vials briefly before placing into thermal cycler.  
*[Note: There should not be any bubbles in the reaction mix. Bubbles interfere with fluorescence detection.]*

**Programming Thermal cycler**

<b>Sample volume</b>	25µl
<b>Fluorescence Dyes</b>	FAM
<b>Passive reference</b>	None
<b>Ramping rate</b>	Default

**Thermal Profile**

	<b>Step</b>	<b>Time</b>	<b>Temp</b>
	Reverse transcriptase	20min	50°C
	Taq enzyme activation / Hold	15min	95°C
<b>45 cycles</b>	Denaturation	20sec	95°C
	Annealing/Data collection*	20sec	60°C
	Extension	20sec	72°C

<b>Data collection/Acquisition</b>	<b>Targets</b>
FAM	RDRF-nCoV-2019 ORF-nCoV-2019

**Reading the graph:**

Select the NTC and Positive control wells, view the graph of amplification. The NTC must be flat with no Ct value. If required adjust the threshold value just above the NTC. Do not report if NTC is amplified. The PC must be amplified. Select test samples wells one by one and view the amplification.

NTC justifies NO contamination in the reagent as well as fine pipetting and its environment. PC justifies the reagents storage conditions and reaction parameters are as prescribed.

**Qualitative interpretation of results:**

NTC	PC	Screening assay	RDRP	ORF	Interpretation
Negative	Positive	Positive	Positive	Positive	Novel Coronavirus-2019 RNA detected
Negative	Positive	Positive	Positive	Negative	Novel Coronavirus-2019 RNA detected
Negative	Positive	Positive	Negative	Positive	Novel Coronavirus-2019 RNA detected
Negative	Positive	Positive	Negative	Negative	SARS-Like CoV RNA detected
Negative	Negative	---	Negative	Negative	Experiment failed – repeat
Positive	Positive	---	Positive	Positive	Experiment failed – Check the reagent or environment contamination

**Limitations**

Good laboratory practice is essential for proper performance of this assay. Strict compliance with the instructions for use is required for optimal results.

Analysts should be trained and familiar with testing procedures and interpretation of results prior to performing the assay.

A false negative result may occur if inadequate numbers of organisms are present in the sample due to improper collection, transport or handling. Appropriate specimen collection, transport, storage and processing procedures are required for the optimal performance of this test.

Extreme care should be taken to preserve the purity of the components of the kit and reaction setups. All reagents should be closely monitored for impurity and contamination. Any suspicious reagents should be discarded.

The presence of PCR inhibitors may cause under quantification, false negative or invalid results.

Potential mutations within the target regions of the pathogen's genome covered by the primers and/or probes used in the kit may result in under quantification and/or failure to detect.

As with any research, the HELINI Coronavirus Real-time PCR results need to be interpreted in consideration of all clinical and laboratory findings.



### Quality Control

In accordance with the HELINI Biomolecules in house Quality Management System, each lot of HELINI Coronavirus Real-time PCR confirmation two tube assay kit is tested against predetermined specifications to ensure consistent product quality.

### Explanations of symbols



Research use only



Catalogue number



Pack size – number of tests



Manufacturer

Manufactured by

**HELINI Biomolecules,**

Ohmlina, 26, 2<sup>nd</sup> Avenue,

Khuthubi Complex, Vettuvankeni,

Chennai 600115, Tamil Nadu, INDIA

www.helini.in

info@helini.in

helinibiomolecules@gmail.com

+91-44-24490433 / +91-9382810333

